

IMPORTANT NOTE:

Clear Results Drug Tests have a temperature check on them. This means that every urine sample will show a temperature reading. The temperature of the urine collected should fall between 32°C to 38°C or 90°F to 100°F.

If the temperature of the urine collected falls outside of this range, this could mean that the urine sample has been altered, tampered with, or substituted.

However, it is normal to have the temperature of the urine be slightly lower after 5 minutes when it is suggested that you read the results.

FREQUENTLY ASKED QUESTIONS:

Q: How do I know if the Clear Results Home Drug Test worked?

A: As long as there is a red line (dark or faded) next to the word CONTROL the test is working.

Q: How soon can I read my results?

A: You can read your results after 5 minutes as long as there is a red line (dark or faded) next to the word CONTROL.

Q: What is the best time of day to take the test?


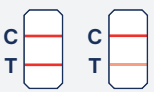
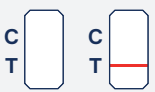
A: Anytime.

Q: What does a preliminary positive result mean?

A: Clear Results Home Drug Tests are meant for at home use. A preliminary positive result still needs to be confirmed by a healthcare professional or doctor.

Q: How to read your results:

A:

		
<p>Preliminary Positive Result (Drug(s) Detected)</p> <ul style="list-style-type: none"> A red line (or faded red line) appears next to the word control (C) ONLY. No line appears next to the word drug (T) indicates a preliminary positive result for that drug. 	<p>Negative Result (No Drug(s) Detected)</p> <ul style="list-style-type: none"> A red line (or faded red line) appears next to the word control (C) AND a red line (or faded red line) appears next to the word drug (T). This is regardless of how dark or faded the red lines appear. 	<p>Invalid Result</p> <ul style="list-style-type: none"> If after 10 minutes NO LINE appears next to the word control, the test is invalid.

If your test shows a POSITIVE RESULT, please contact your health care professional or doctor to confirm test results.

If you have any questions, please visit our website at www.clearresultsdrugtest.com


Clear Results Home Drug Tests provide you with fast, accurate and confidential drug testing.

Our tests can test for up to 14 different drugs based on which test you have purchased. Each test kit is for an individual sample to be tested and is not intended for reuse.

BEFORE YOU TAKE THE HOME DRUG TEST


- Read these instructions completely and ensure you understand them.
- Ensure you have a way to check the time.
- Ensure that the test is NOT expired. Always check the expiry date first and if it is expired, DO NOT use the product.
- In order to take the test, be sure you have: test collection cup, information insert (this page), lid.
- Prior to use, ensure that the test was sealed in the foil pouch.

Step 1 Take the test



A. Remove the test from the sealed foil pouch.
B. Collect urine sample in cup.

Step 2 Seal test to begin testing process

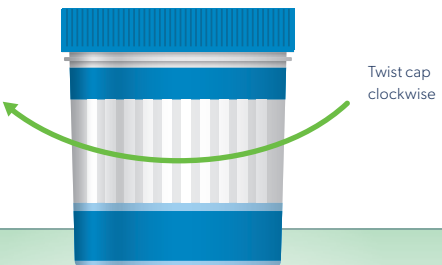



Securely attach the lid to the test collection cup and set on a flat surface.

Step 3 Read the results

Wait for 5 minutes after you have taken the test. Peel back the label to reveal the test results.

A red line (or faded red line) must appear next to the word CONTROL **(C)**. This will let you know that the test has worked properly.

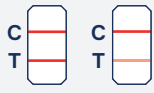
<p>A) Negative Result (No Drug(s) Detected)</p> <ul style="list-style-type: none"> A red line (or faded red line) appears next to the word control (C) AND a red line (or faded red line) appears next to the word drug (T). This is regardless of how dark or faded the red lines appear.

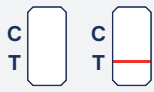
<p>B) Preliminary Positive Result (Drug(s) Detected)</p> <ul style="list-style-type: none"> A red line (or faded red line) appears next to the word control (C) ONLY. No line appears next to the word drug (T) indicates a preliminary positive result for that drug.

<p>C) Invalid Result</p> <ul style="list-style-type: none"> If after 10 minutes NO LINE appears next to the word control (C), the test is invalid.

Clear Results

UNDERSTANDING YOUR RESULTS


Depending on the Clear Results Home Drug Test Kit you purchased, one of the following six options applies:

14 DRUGS TESTED




- Marijuana (**THC**)
- Cocaine (**COC**)
- Opiates (**OPI**)
- Methamphetamine (**MET**)
- Amphetamines (**AMP**)
- Ecstasy (**MDMA**)
- Phencyclidine (**PCP**)
- Tricyclic Antidepressants (**TCA**)
- Barbiturates (**BAR**)
- Benzodiazepines (**BZO**)
- Methadone (**MTD**)
- Oxycodone (**OXY**)
- Fentanyl (**FEN**)
- Hydromorphone (**HYD**)

8 DRUGS TESTED




- Marijuana (**THC**)
- Cocaine (**COC**)
- Opiates (**OPI**)
- Methamphetamine (**MET**)
- Amphetamines (**AMP**)
- Ecstasy (**MDMA**)
- Phencyclidine (**PCP**)
- Fentanyl (**FEN**)

5 DRUGS TESTED

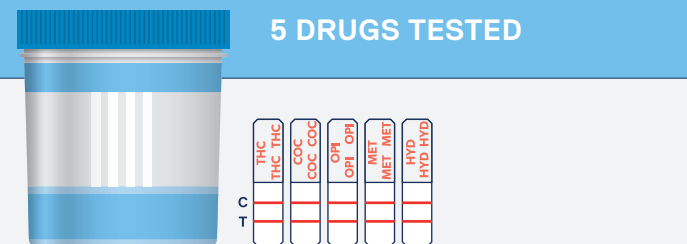


- Marijuana (**THC**)
- Cocaine (**COC**)
- Opiates (**OPI**)
- Methamphetamine (**MET**)
- Hydromorphone (**HYD**)

2 DRUGS TESTED



- Marijuana (**THC**)
- Cocaine (**COC**)

1 DRUG TESTED

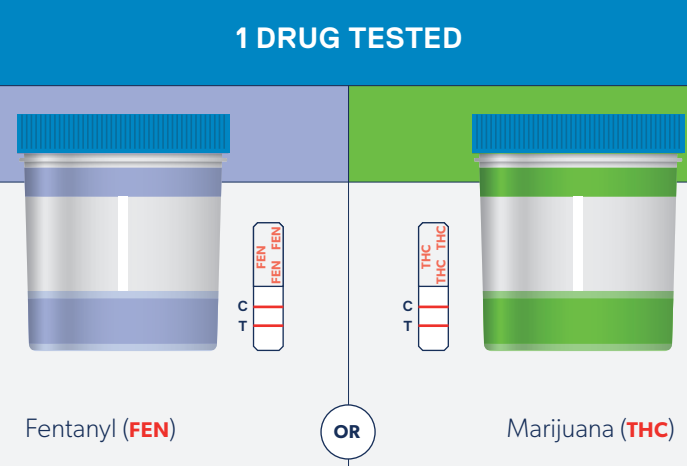

Fentanyl (**FEN**) OR Marijuana (**THC**)

If your test shows a POSITIVE RESULT, please contact your health care professional or doctor to confirm test results.

If you have any questions, please visit our website at www.clearresultsdrugtest.com

Clear Results Home Drug Tests provide you with fast, accurate and confidential drug testing.

Our tests can test for up to 14 different drugs based on which test you have purchased. Each test kit is for an individual sample to be tested and is not intended for reuse.

DRUG/IDENTIFIER	CUT-OFF LEVEL	MINIMUM	MAXIMUM
Marijuana (THC)	50 ng/ml	2 hours	Up to 40+ days
Cocaine (COC)	300 ng/ml	1–4 hours	2–4 days
Methamphetamine (MET)	1000 ng/ml	2–7 hours	2–4 days
Amphetamines (AMP)	1000 ng/ml	2–7 hours	2–4 days
Ecstasy (MDMA)	500 ng/ml	2–7 hours	2–4 days
Opiates (Heroin) (OPI)	2000 ng/ml	2 hours	2–3 days
Phencyclidine (PCP)	25 ng/ml	4–6 hours	7–14 days
Benzodiazepines (BZO)	300 ng/ml	2–7 hours	1–4 days
Methadone (MTD)	300 ng/ml	3–8 hours	1–3 days
Barbiturates (BAR)	300 ng/ml	2–4 hours	1–3 weeks
Tricyclic Antidepressants (TCA)	1000 ng/ml	8–12 hours	2–7 days
Oxycodone (OXY)	100 ng/ml	1–3 hours	1–2 days
Fentanyl (FEN)	1-5 ng/ml	1–3 hours	1–2 days
Hydromorphone (HYD)	100 ng/ml	1–3 hours	1–2 days